


ONETOUCH Reveal®

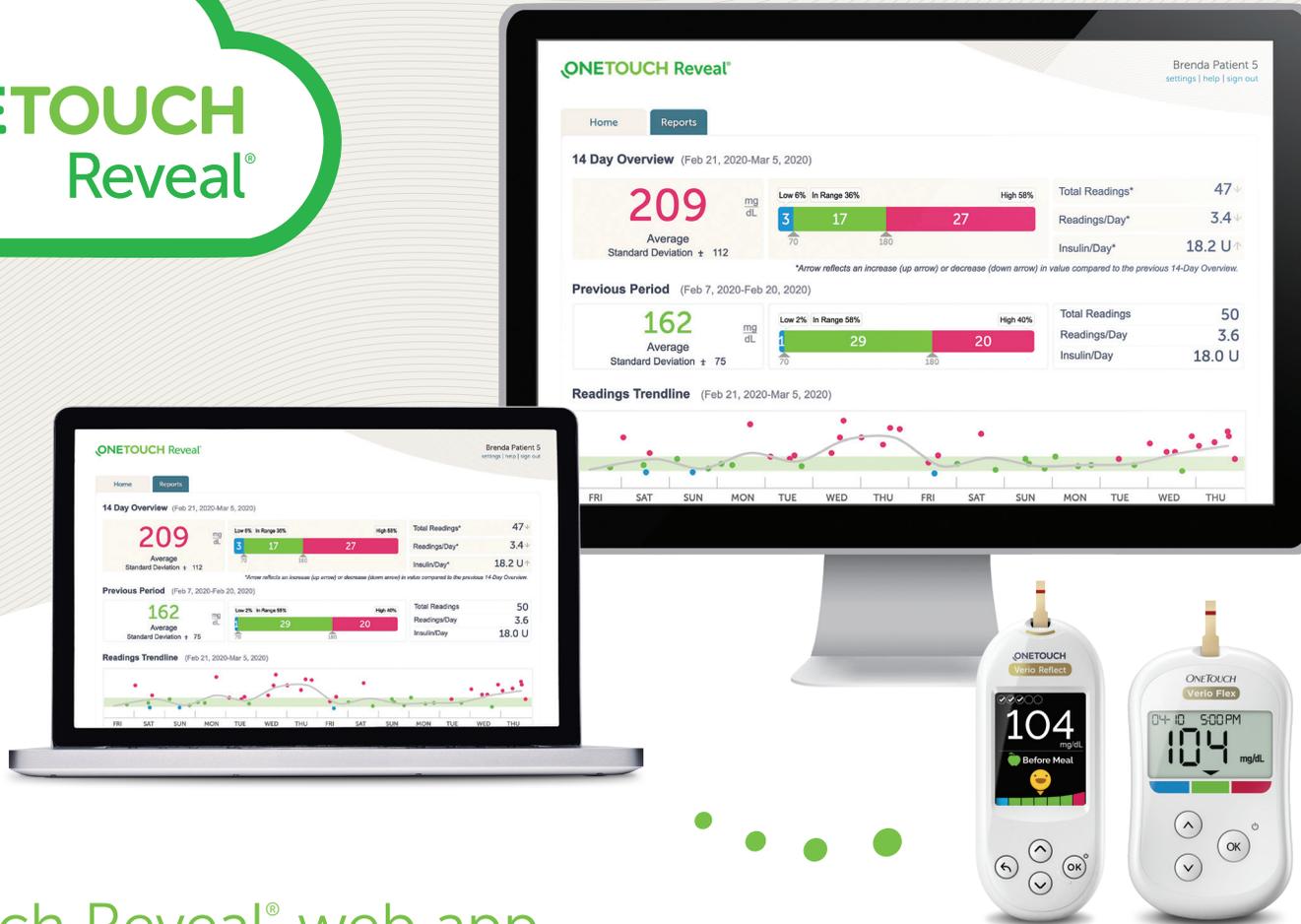

OneTouch Reveal® web app

Helps your doctor see more about your blood sugar.*

- Tracks your blood glucose, food and insulin dosing over time
- Reports can be printed, ready for discussion at your next appointment with your healthcare professional
- Can be used on PC and Mac computers at [OneTouchReveal.com](https://www.onetouchreveal.com)

ONETOUCH®
every touch is a step forward®

* Based on a survey of HCPs comparing the use of OneTouch Reveal® reports with the use of paper logbooks.

How to setup your OneTouch Reveal® web app

Before you begin, confirm that cookies are enabled in your browser.



Wireless Option

OR

Cable Option

1. Connect wireless devices and activate your account

- Install the OneTouch Reveal® mobile app on your mobile device
- Open the mobile app and follow the on-screen instructions to setup your account and pair your OneTouch Verio Reflect® or OneTouch Verio Flex® meter

For more details, refer to the OneTouch Reveal® mobile app **Help File**.

2. Sign in to the OneTouch Reveal® web app on your computer

- Go to **OneTouchReveal.com** and enter the same **Username** and **Password** that you used in the OneTouch Reveal® mobile app
- Click **Sign In** and follow the on-screen instructions

1. Install Data Transfer Tool

- Unplug all OneTouch® USB cables from your computer
- **Sign In** to your OneTouch Reveal® account and click **help** from the top right of any screen
- Click the **Download** button under **Data Transfer Tool** and follow the prompts
- Select the appropriate devices
- Click **Finish**. After installation, you may be prompted to re-start your computer

2. Transfer data from your OneTouch® meter

- Connect your meter to the computer with your OneTouch® USB cable (*cable sold separately*)
- Open the OneTouch Reveal® Data Transfer Tool and select **Detect Now**. When the device is found, a confirmation screen will appear
- **Sign In** to the OneTouch Reveal® web application
- Click **Confirm** to transfer data. When the transfer completes, click **OK**




You're all set! You can now access your data with the OneTouch Reveal® web app at **OneTouchReveal.com**

ONETOUCH®
every touch is a step forward®



View your blood glucose readings

- Click the **Home** tab to see your blood glucose readings
- You can see your color-coded blood glucose readings, **High (red)**, **In Range (green)**, **Low (blue)**, over the previous 14 days.

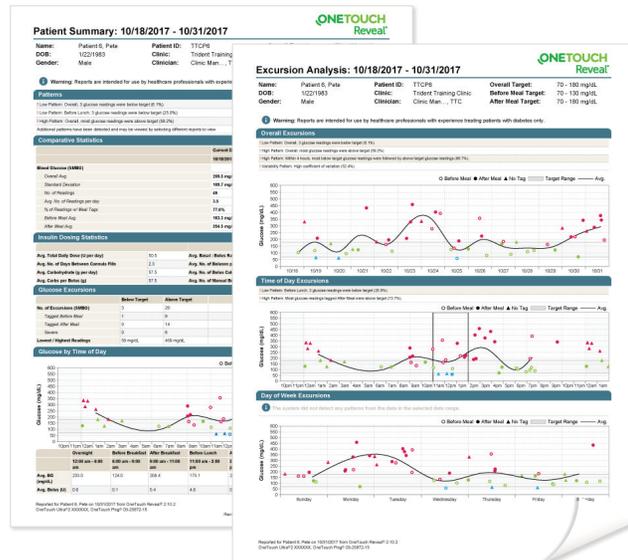

Prepare for your next visit with your healthcare professional

Select the **Report** you want to discuss with your healthcare professional. Click **Download Report**.

Any questions? We're here to help



Visit [OneTouch.com](https://www.onetouch.com)



Call us at 1-866-693-0599

Find the necessary system requirements at
[OneTouchReveal.com/support/en_US/](https://www.onetouch.com/support/en_US/)

